



yuwell

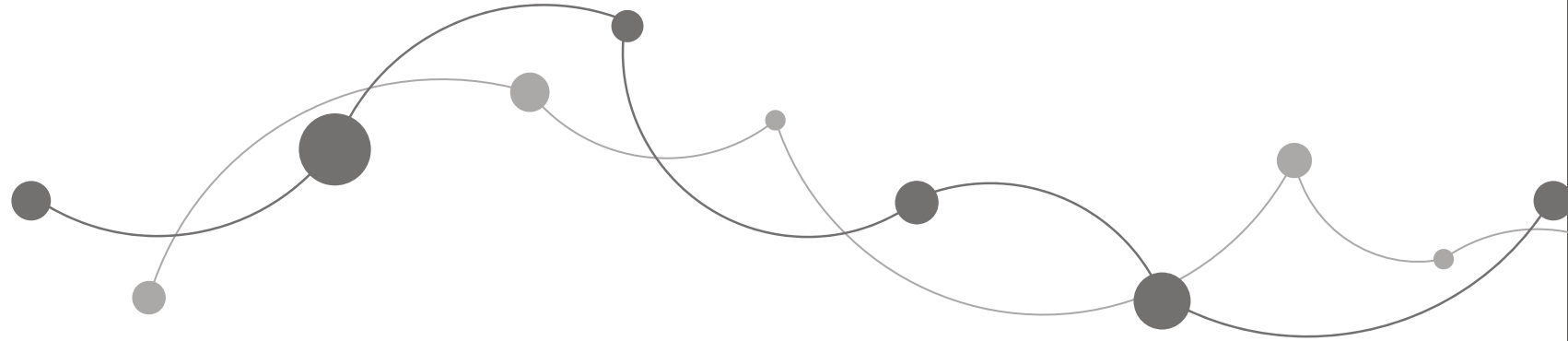

We reserve the right to change the technology and appearance of the products. If there is any change, we will not inform you.

Registrant/Production/After-sales service unit:
Jiangsu Yuyue Medical Equipment & Supply Co.,Ltd.
Location/Production Address: Yunyang Industrial
Park,Danyang City, Jiangsu Province, P.R China

430508-0A 

Anti-decubitus Beds Cushion

Specifications:

Round Type B/Square Type B/Back Type A

User's Manual

Please read the instructions carefully before using the product.

Suitable Range

- ▶ Used for preventing decubitus or relieving fatigue of bottom, back and other parts of postoperative or bedridden patients.
- ▶ No contraindication.

Performance Structure and Composition

- ▶ This product consists of air cushion and nozzle.

Main Technical Indicators

- ▶ Max load of round and square type cushion: 100kg.
- ▶ Size after expansion:

Type	Basic size(mm)		
	Length	Width	Height
Round type B	460 ± 10 (External diameter)	120 ± 10 (Internal diameter)	85 ± 5
Square type B	440 ± 10	440 ± 10	85 ± 5
Back type A	580 ± 10	420 ± 10	/

- ▶ Symbols and meanings of security requirements in this product




	Upward		Fragile		Keep Dry
---	--------	---	---------	---	----------

Usage

- ▶ This product can be inflated by using inflator aiming the nozzle or with other methods.
- ▶ This product shall be placed in the middle of the chair and limited to single use.

Maintenance and precautions

- ▶ Warnings and attention are shown here for proper and safe use of products to prevent damage or damage to users or others. Warning and attention are as follows:

Figure	Content
	It is possible to cause injury or damage to the product when it is used wrongly.
	Ⓢ Means compulsion or something must be obeyed. See contents in or around Ⓢ For details. Left image means 'general coercion'
	Ⓢ Means prohibition which means something is not allowed. See contents in or near Ⓢ for details. Left image means 'general prohibition'.

- Ⓢ People who weigh more than 100kg should not use round and square cushions.
- Ⓢ Control the pressure to prevent the filling of the air bag when inflating.
- Ⓢ Do not touch sharp substances to prevent rupture and puncture of air bags to cause air leakage.
- Ⓢ The product should be kept clean and hygiene. Neutral detergent is often used to clean the surface.
- ⚠ Attention: away from acid and alkali.
- Ⓢ The product is placed in a dry, ventilated and non corrosive gas room.
- ▶ The disposal of wastes should be handled according to the relevant environmental protection regulations of the government.
- ▶ Product life time: 1 year.
- ▶ Appendix list: Manual 1PC, certificate 1PC.

After-sale service

- ▶ Please verify the airtightness and surface quality when buying. The product will not be returned once purchased.