

Instruction for use of Disposable Blood Lancets(soft)

[Product Name] Disposable Blood Lancets

[Type] Soft

[Specifications] 21G,23G,26G,28G,30G,32G,33G

[Intended purpose] For capillary blood sampling

[Caution]

- The lancet tip is guaranteed sterile while the protective disk is sealed to the base.
- Do not use if the seal has been damaged or broken.
- Do not store lancet in lancing devices.
- Please keep the product away from the children.
- The lancet is disposable and never use it more than once or share with others.

[Contraindications] Unknown.

[Instructions for Use]



1. Load your lancing device and twist off the protective cap.
2. Consult the Instructions for use of your lancing device.
3. After use, please dispose of the used lancet in an approved sharps container according to the local sharp disposal regulations.

[Symbolic interpretation]



Sterilized using irradiation



Do not re-use



Lot number



Use-by date



Manufacturer



European Authorized Representation



Notified Body



Caution

[Notified Body]

TÜV Rheinland LGA Products GmbH
Tillystraße 2, 90431, Nürnberg, Germany

[European Authorized Representation]

EMERGO EUROPE
Prinsessegracht 20,2514 AP, The Hague,The Netherlands

[Manufacturer]

SteriLance Medical (Suzhou) Inc .
No.168 PuTuoShan Road, New District,215153 Suzhou,Jiangsu ,P.R.China

[Revised date] September 22, 2019 (Version 02)