


yuwell

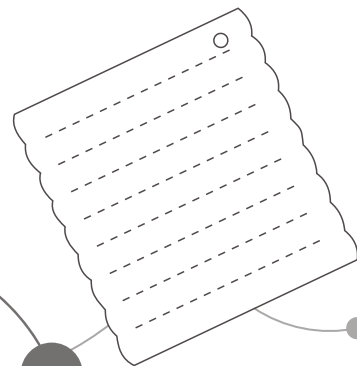

Anti-decubitus Seat(square)

User's Manual

JIANGSU YUYUE MEDICAL EQUIPMENT & SUPPLY CO.,LTD.
Sales Center Address: Huanyuan East Road No.1,Xuzhuang
Software Park, Nanjing, Jiangsu Province, P.R. China, 210000
Manufacturing Address: YunYang Industrial Park
DanYang, Jiangsu Province, P.R. China, 212300
<http://www.yuwell.com>


430405-0A 

Please read the user's manual closely before using!

SUMMARY

- ▶ Portable, lightweight, compact , easy to use. The unique wavy design makes air ventilation, keeps skin dry and promotes blood circulation.
- ▶ This product is suitable for all kinds of people, including those who need rehabilitation therapy and long term office workers.

STRUCTURE DIAGRAM

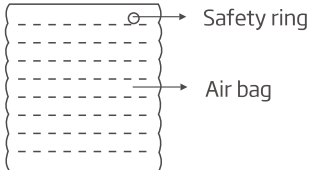- ▶ Model: Square  Safety ring
Air bag

MAIN SPECIFICATION

- ▶ The anti-decubitus seat (square) is made of 700um film flocking or PVC coating material.
- ▶ Weight capacity: 100kg
- ▶ Dimension after expansion:

Length	Width	Thickness
450mm	420mm	55mm

USAGE METHOD

- ▶ The product can be inflated by a hand pump through the safety ring.
- ▶ The product should be placed in the middle of a seat and only for one person.

MAINTANENCE&ATTENTION

Marking	Description
⚠	INCORRECT USE, may damage the product or hurt person.
ⓘ	Means ENFORCEMENT and must be followed.
⊘	Means FORBIDDEN and must not do.

- ⓘ The anti-decubitus seat (square) is not suitable for person over 100kg.
- ⓘ Be careful when inflating to prevent the air bag from breaking.
- ⊘ Keep away from sharp objects to prevent being scratched.
- ⓘ Clean the product regularly with neutral detergent.
- ⚠ Note: acid prevention, alkali prevention
- ⓘ The product should be placed in dry and ventilating room without corrosive gas.
- ▶ The castoff should be disposed in accordance with all applicable government regulations.

All specifications and product configurations are subject to change without notification.